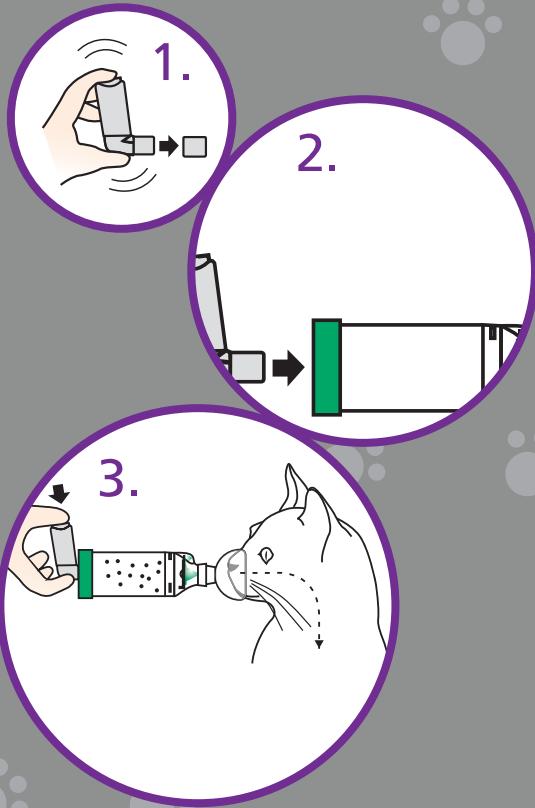
## Instructions for Use

1. Remove cap and shake inhaler
2. Insert inhaler into back of chamber and gently apply the mask to your cat's face
3. Press the inhaler to release the medication when the **Flow-Vu\*** Inhalation Indicator is moving toward your cat and hold the mask in place for 5–6 breaths

NOTE: The mask should cover your cat's nose and mouth but not the eyes.

## Tips and tricks

Here are some tips and tricks from some other **AeroKat\*** Chamber users.

- ▶ Reward your cat with treats before and/or after treatment
- ▶ Swaddle your cat in a towel or blanket
- ▶ Slowly introduce the **AeroKat\*** Chamber. Start with the mask only, getting your cat used to the feel of it
- ▶ Use the **Flow-Vu\*** Inhalation Indicator to help ensure your cat has inhaled their medication
- ▶ Reference the numerous videos on YouTube
- ▶ Be patient — don't give up!

725 Third Street, London,  
Ontario, Canada, N5V 5G4  
+1-519-455-7060 F: +1-519-455-7858  
1-800-465-3296  
[www.aerokat.com](http://www.aerokat.com)

101006-001 Rev. D \* Trade-marks and registered trademarks of Trudell Medical International. ©T.M. 2015. All rights reserved.

# Does your cat suffer from asthma symptoms?

## If your cat experiences any of the following symptoms, they may be suffering from feline asthma:

- ▶ Frequent wheezing
- ▶ Persistent coughing
- ▶ Open mouth or labored breathing

Approximately one in 100 cats has asthma – perhaps more, as the symptoms are often confused with hairballs. Asthma usually affects cats between 2 and 8 years of age with Siamese and Himalayan breeds having a higher prevalence.

## Treatment

The **AeroKat\*** Feline Aerosol Chamber is designed to deliver Metered Dose Inhaler medication as recommended by a veterinarian to cats with feline asthma, chronic bronchitis or allergic rhinitis.

When the **AeroKat\*** Chamber is used with the inhaler it allows your cat to breathe normally and inhale the medication into their lungs, where it is needed.

Can be used with all types of Metered Dose Inhalers

“Works like a charm — the mask fits *purr*-fectly over the nose/mouth and chamber holds in the meds to inhale. It even includes an indicator to show how many breaths are taken.

Highly recommend !!!”

Cat Owner

Feline Aerosol Chamber

### Flow-Vu\* Inhalation Indicator

Takes the guesswork out of administering your cat's inhaler medication. The **Flow-Vu\*** Inhalation Indicator moves toward your cat as they inhale and will only move if there is a good seal.

### Inhalation Valve

Extra sensitive, low resistance valve is designed to respond to shallow, low volume breaths

### Anti-Static Chamber

Holds the medication allowing your cat time to inhale

### 2 Mask Sizes

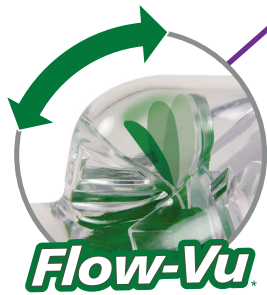
Each device is shipped with 2 different sized facemasks. The masks are specifically designed to reduce irritation to your cat's fur.

The **AeroKat<sup>®</sup>** Feline Aerosol Chamber is designed to be used with a Metered Dose Inhaler (puffer) to deliver aerosol medication to cats with feline asthma, chronic bronchitis or allergic rhinitis.

The use of inhaled medication provides targeted delivery of medication to the lungs and reduces the side effects seen in cats treated with systemic medications.<sup>1</sup> Therapeutic effect can often be achieved with only a fraction of the dose required for systemic delivery of the same drug.<sup>2</sup>

### **Flow-Vu<sup>®</sup>** Inhalation Indicator

The **Flow-Vu<sup>®</sup>** Indicator moves with inhalation and provides the caregiver with visual confirmation of the cat inhaling their medication. This is important because some cats hold their breath when first introduced to the mask. The **Flow-Vu<sup>®</sup>** Indicator also allows the caregiver to count breaths and ensure a proper facemask seal.

### **2 Mask Sizes**

Each device is shipped with 2 different sized facemasks. The mask should cover the cat's nose and mouth but not the eyes.

### **Inhalation Valve**

Extra sensitive, low resistance valve is designed to respond to shallow, low volume breaths

### **Anti-static Chamber**

Holds the medication allowing the cat more time to inhale

### **Universal Backpiece**

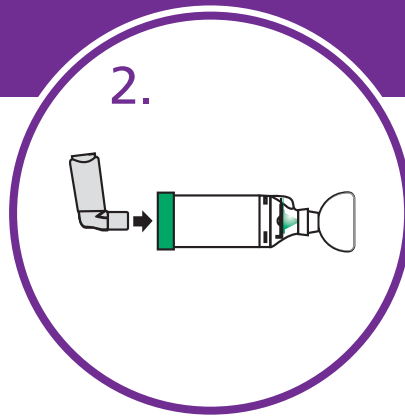
Can be used with all commonly prescribed Metered Dose Inhalers  
(Note: not designed for use with Dry Powder Inhalers)


## Easy to Use

1. Remove cap and shake inhaler

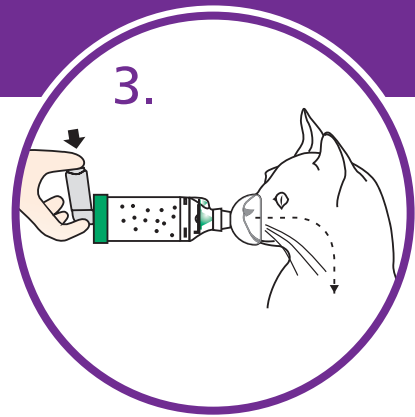2. Insert inhaler into back of chamber and gently apply the mask to the cat's face

3. Press the inhaler to release the medication when the **Flow-Vu\*** Inhalation Indicator is moving toward the cat and hold the mask in place for 5–6 breaths

## Metered Dose Inhaler Treatment

Adapted from: Padrid P. *Use of inhaled medications to treat respiratory diseases in dogs and cats*. Journal of the American Animal Hospital Association 42:165-169 (2006).

DRUG	DOSE	USAGE	INDICATION
<b>Salbutamol</b> <ul style="list-style-type: none"> <li>Available through different manufacturers</li> <li>Single uniform strength: 100 µg/actuation</li> </ul>	<ul style="list-style-type: none"> <li>1-2 puffs (100 µg/puff) administered via <b>AeroKat*</b> Chamber</li> <li>Each Metered Dose Inhaler contains 200 puffs</li> </ul>	<ul style="list-style-type: none"> <li>Use as needed, but not routinely</li> <li>In emergency cases, albuterol can be used q30 minutes for up to 4 to 6 hours without serious side effects</li> </ul>	Immediate relief of bronchoconstriction
<b>Fluticasone Propionate</b> <ul style="list-style-type: none"> <li>Brand name: Flixotide<sup>†</sup> Evohaler<sup>†</sup> Metered Dose Inhaler</li> <li>Available in three strengths: 50 µg, 125 µg and 250 µg/actuation</li> </ul>	<ul style="list-style-type: none"> <li>1 puff (125 µg) q12h administered via <b>AeroKat*</b> Chamber</li> <li>Each Metered Dose Inhaler contains 120 puffs</li> <li>Cats with more serious disease may require 250 µg q12h<sup>4</sup></li> </ul>	Overlap with oral prednisolone for 2-3 weeks when starting since Fluticasone <sup>†</sup> HFA takes 1-2 weeks to reach peak effect	Long-term control of airway inflammation

### NOTES:

- If the cat gets frightened when pressing the inhaler, press the inhaler before placing the mask on the cat's face.  
Note: this means the cat will receive a lower dose and may then require a second administration to guarantee that it receives an adequate amount of medication for treatment.<sup>3</sup>
- If the cat is treated with both fluticasone and salbutamol, the salbutamol should be given first.<sup>3</sup>
- Some of the newer Metered Dose Inhalers include dose indicators. If not, owners are encouraged to keep track of puffs.

<sup>1</sup> Padrid P. *Use of inhaled medications to treat respiratory diseases in dogs and cats*. Journal of the American Animal Hospital Association 42:165-169 (2006).

<sup>2</sup> Labiris NR, Dolovich MB. *Pulmonary drug delivery. Part II: the role of inhalant delivery devices and drug formulations in therapeutic effectiveness of aerosolized medications*. Br J Clin Pharmacol 2003;56(6):600-612.

<sup>3</sup> Pernas GS. *Feline asthma*. Veterinary Focus. Vol 20 no 2, 2010: 10-17.

<sup>4</sup> Padrid P. *Feline bronchial disease including use of inhaled medications*. Proceedings of the Southern European Veterinary Conference, 2010.

MD-633A-1215 \* trade-marks and registered trade-marks of Trudell Medical International. † trade-marks and registered trade-marks of the respective companies. © TMI 2008, 2010, 2015. All rights reserved.

Trudell Medical International<sup>†</sup>

725 Third Street, London,  
Ontario, Canada, N5V 5G4

Distributed by: **Breatheazy**<sup>®</sup>  
respiratory relief for cats, dogs and horses

154 Worcester Road, Malvern, Worcestershire WR14 1AA,  
United Kingdom Tel: +44 845 680 8975 Fax: +44 845 680 8976  
enquiries@breatheazy.co.uk

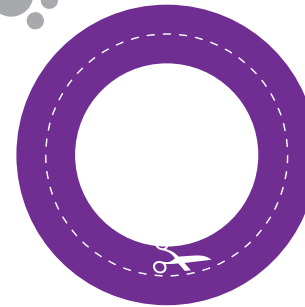
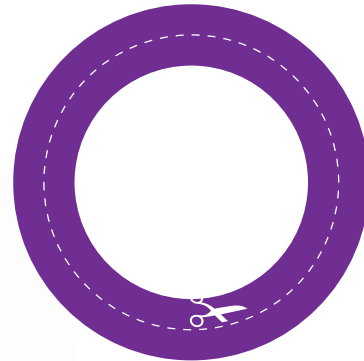
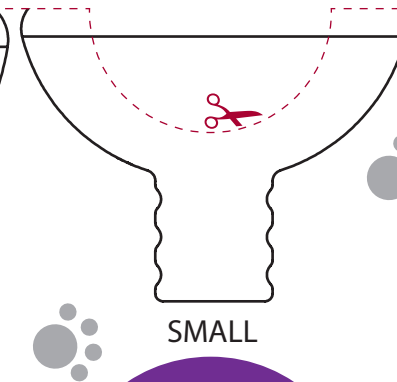
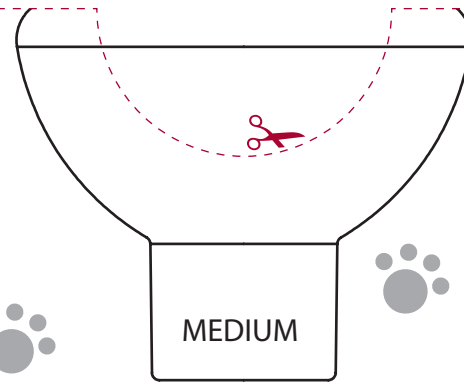
**AeroKat** 

Feline Aerosol Chamber

www.aerokat.co.uk

**HOW TO USE:** Print this template on letter paper, landscape. DO NOT SCALE. Print at 100% or disable page scaling. Cut along the dashed lines and remove inner shape. Insert your cat's nose and mouth into the front and side cut-outs to see which mask will fit best.

**AeroKat**   
Feline Aerosol Chamber

The exclusive **Flow-Vu**<sup>®</sup> Inhalation Indicator moves toward the cat as they inhale and will only move if there is a good seal.

## Mask Sizer

Each **AeroKat**<sup>®</sup> Chamber is shipped with a small and medium mask. The mask selected should cover the cat's nose and mouth but not the eyes.

Helping animals around the world breathe easier

Trudell Medical International<sup>®</sup>

[www.aerokat.com](http://www.aerokat.com)

# AeroKat

Feline Aerosol Chamber
www.aerokat.com

#### EN INTENDED USE

The chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to your cat or dog's lungs as prescribed by your veterinarian.

#### BEFORE USE

The chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read. Carefully examine the chamber and replace immediately if any defect is noticed. The chamber is shipped with 2 different sized masks. Select the mask that fits best and insert on frontpiece of chamber. The mask should cover the animal's nose and mouth but not eyes.

#### USAGE INSTRUCTIONS

**1** Remove cap and shake the inhaler immediately before use as per the instructions supplied with it.
**2** Insert the inhaler into the backpiece of the chamber and gently apply the mask to your animal's nostrils while gently holding the muzzle closed.
**3** Press the inhaler to release the medication. Hold the mask in place for 5-6 breaths as the cat or dog inhales the medicine.

#### TIPS AND TRICKS:

• Slowly introduce the chamber. Start with the mask only, getting the animal used to the feel of it
• Reward the animal with treats before and / or after treatment
• Swaddle them in a towel or blanket
• If your cat or dog does not respond well to the sound of the inhaler, hold the chamber behind your back and press the inhaler and then immediately place mask on their face.

<b>Notes:</b>
• Use the <i><b>Flow-Vu</b></i> * Inhalation Indicator to ensure a good seal, coordinate pressing the inhaler with inhalation and to count the number of breaths taken. The <i><b>Flow-Vu</b></i> * Indicator moves toward your cat or dog as they inhale and only moves if there is a proper seal.
• Administer one (1) puff at a time. Follow instructions supplied by your veterinarian on the number of puffs to take and how long to wait before repeating.
• Wipe the animal's face with a damp cloth after treatment to remove medication residue.

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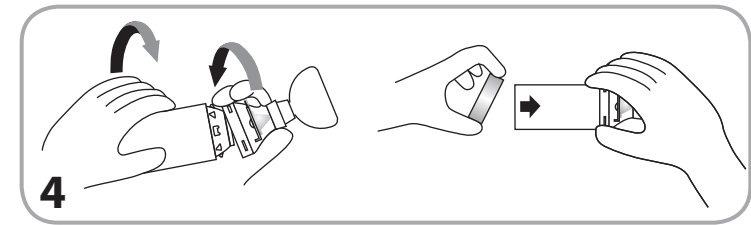
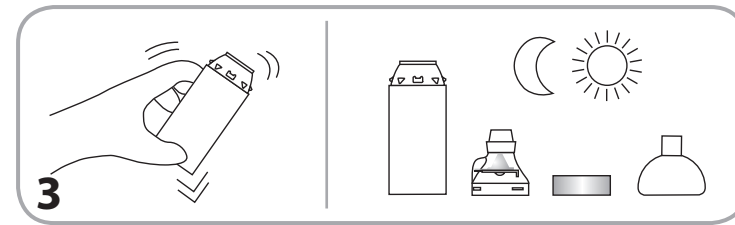
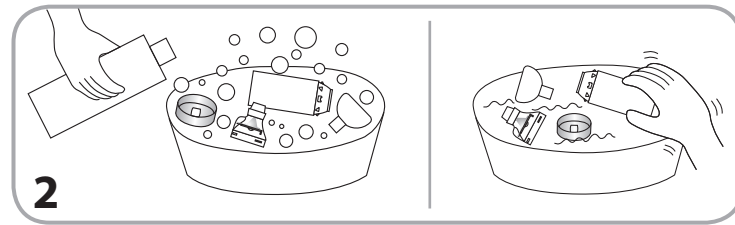
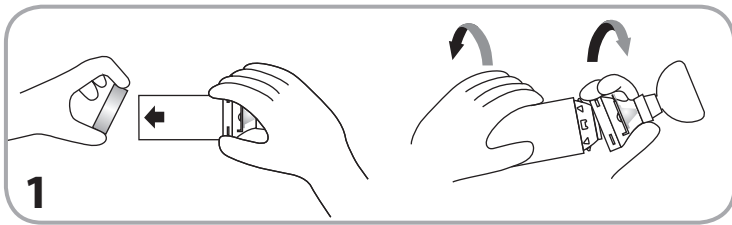
## EN CLEANING INSTRUCTIONS

The chamber should be cleaned weekly.

**1** Remove the backpiece (A). To detach the frontpiece (B), twist chamber as shown and then remove mask (C) from frontpiece. **2** Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently and then rinse parts in clean water. **3** Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly. **4** To reassemble, fit the mask on the frontpiece and then place frontpiece on the end of the chamber and twist firmly until securely locked into position. Center the Alignment Feature (D) on the backpiece with the **Flow-Vu\*** Indicator (E) as shown. Press firmly to attach the backpiece.

### Notes

- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Do not boil, sterilize or clean in a dishwasher.
- Product should be replaced after 12 months of continuous use

### Cautions

- Do not leave the chamber unattended with children.

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